# PRODUCT SPECIFICATION SHEET



# Ciprofloxacin (AS018-1vl)

# Intended Use

Ciprofloxacin CIP 5 mcg discs are used for antimicrobial susceptibility testing of bacterial cultures as per Bauer-Kirby Method.

# Product Summary and Principle:

Antibiotic sensitivity is the susceptibility of bacteria to antibiotics. Antibiotic susceptibility testing (AST) is usually carried out to determine which antibiotic will be most successful in treating a bacterial infection *in vivo*. Antimicrobial susceptibility testing (AST) of bacterial and fungal isolates is a common and important technique in most clinical laboratories. The results of these tests are used for selection of the most appropriate antimicrobial agent(s) for treatment against the infectious organisms. Till the 1950s, laboratories were lacking in the methodologies and equipments for the accurate determination of in vitro responses of organisms to antimicrobial agents. Bauer et al<sup>(1)</sup> began the development of standardized methods for antimicrobial susceptibility testing, using disc diffusion system. However the susceptibility results may not always correlate with the patient's response to therapy. The response of an infected patient to antimicrobial agent(s) is a complex interrelationship of host responses, drug dynamics and microbial activity.

Antimicrobial susceptibility tests are either quantitative or qualitative. Disc diffusion test is a qualitative test method. The National Committee for Clinical Laboratory Standards (NCCLS), now known as Clinical Laboratory Standards Institute (CLSI) has published comprehensive documents regarding the disc diffusion systems. The agar disc diffusion test is the most convenient and widely used method for routine antimicrobial susceptibility testing. In subsequent and current practice, antimicrobial impregnated paper discs are applied onto the agar surface. Based on the Bauer-Kirby Method, standardized reference procedures for the disc systems were published by WHO and FDA and are periodically updated by the CLSI (formerly NCCLS)<sup>(2)</sup>. For any antimicrobial testing, Quality control or clinical testing, the method to be followed is the same as mentioned below.

#### Composition:

Ingredients	Concentration
Ciprofloxacin	5mcg/disc

#### **Precautions:**

- 1. For Laboratory Use only.
- 2. On receipt the discs are to be immediately stored at the recommended temperature.
- 3. Medium preparation, Inoculum preparation and incubation to be done as specified.

# Quality Control Specifications

Appearance: Filter paper discs of 6 mm diameter with printed "CIP" on each side of the disc.

**Expected Cultural Response**: Average diameter of zone of inhibition observed on Mueller Hinton Agar (DM172) after 18 hours incubation at 35-37°C for standard cultures.

Sr. No.	Organisms	Standard zone Diameter in mm
1.	E.coli ATCC 25922	30-40
2.	S. aureus ATCC 25923	22-30
3.	P. aeruginosa ATCC 27853	25-33
4.	H.influenzae ATCC 49247	34-42
5.	N.gonorrhoeae ATCC 49226	48-58

#### Test Procedure





# PRODUCT SPECIFICATION SHEET

- 1. Prepare plates with Mueller Hinton Agar (DM172) for rapidly growing aerobic organisms as per Bauer-Kirby Method. The medium in the plates should be sterile and should have a depth of about 4 mm.
- 2. Inoculate 4-5 similar colonies with a wire, needle or loop to 5 ml Tryptone Soya Broth (DM277) and incubate at 35-37°C for2-8 hours until light to moderate turbidity develops. Compare the inoculum turbidity with that of standard 0.5 McFarland (prepared by mixing 0.5 ml of 1.175% barium chloride and 99.5 ml of 0.36N sulfuric acid). Dilute the inoculum or incubate further as necessary to attain comparative turbidity. Alternatively, the inoculum can be standardized by other appropriate optical method (0.08 0.13 OD turbid suspension at 625 nm)
- 3. Dip a sterile non-toxic cotton swab on a wooden applicator into the standardized inoculum and rotate the soaked swab firmly against the upper inside wall of the tube to express excess fluid. Streak the entire agar surface of the plate with the swab three times, turning the plate at 60° angle between each streaking. Allow the inoculum to dry for 5 15 minutes with lid in place.
- 4. Apply the discs using aseptic technique. When using cartridges, the discs can be applied using the specially designed applicator. When the vials are used, apply the discs using sterile forceps.
- 5. Deposit the discs with centers at least 24 mm apart. For fastidious organisms and for Penicillins and Cephalosporins, the discs should preferably be deposited with centers 30 mm apart.
- 6. Incubate immediately at 35 ± 2°C and examine after 16-18 hours or longer, if necessary. For fastidious organisms incubate at appropriate temperature and time.
- 7. Measure the zones showing complete inhibition and record the diameters of the zones to the nearest millimeter using a calibrated instrument like zone scales.

# Storage

Discs should always be stored at -20°C to +8°C under dry conditions, along with the dessicator provided in individual pack.

# Expiration

Refer to the expiration date stamped on the container. Expiry applies to discs vial when stored as directed. **Packaging** 

Product Name : Ciprofloxacinl Discs (CIP) Product Code : AS018 Available Pack sizes : 1vial

# References

- 1. Bauer, Kirby, Sherris and Turck, 1966, Am. J. Clin. Path., 45: 493
- 2. Performance standards of Antimicrobial Disc Susceptibility Tests, CLSI Vol. 32 No.3, Jan 2012.

# Further Information

For further information please contact your local MICROMASTER Representative.



MICROMASTER LABORATORIES PRIVATE LIMITED Unit 38/39, Kalpataru Industrial Estate, Off G.B. Road, Near 'R-Mall', Thane (W) - 400607. M.S. INDIA. Ph: +91-9320126789/9833630009/9819991103 Email: <u>sales@micromasterlab.com</u>

# Disclaimer:

All Products conform exclusively to the information contained in this and other related Micromaster Publications. Users must ensure that the product(s) is appropriate for their application, prior to use. The information published in this publication is based on research and development work carried out in our laboratory and is to the best of our knowledge true and accurate. Micromaster Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are intended for laboratory, diagnostic, research or further manufacturing use only and not for human or animal or therapeutic use, unless otherwise specified. Statements included herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.



AS018PSS, QAD/FR/024, Rev.00